


Nox Nasal Pressure Cannula with Filter

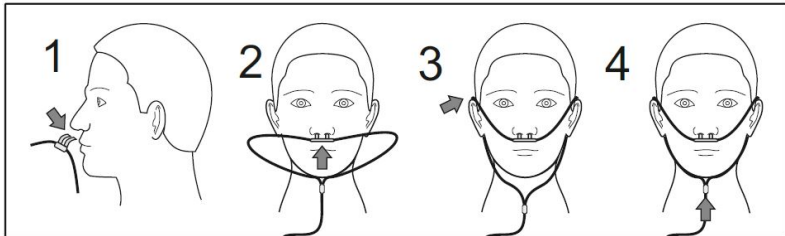

Instructions for use: Place the cannula as illustrated in the figures above.
The nasal prongs should point downward inside the nostrils as shown in step 1.

- Adult Size - 1 unit
- Nox Pressure Connector
- Total Length: 90cm/35in
- Single Use/Disposable
- Clean, Ready for Use
- Caution: Do Not Sterilize

Material information:

- Nasal Prongs: PVC (plastisol, liquid PVC)
- Tube: FPVC
- Filter tube connector: PTFE, TPU, PP
- Luer-lock: ABS

Operation: +5°C to +50°C (+41°F to +122°F)
Storage: -25°C to +70°C (-13°F to +158°F)

Disposal: Follow local governing ordinances and recycling instructions regarding disposal or recycling of this device.



(01)15694311110231
(11)200430
(10)042016N56



www.noxmedical.com/ifu

CE 2797



042016N56



2020-04-30



Manufactured for:

Nox Medical
Katrínartúni 2
IS-105 RVK
Iceland

Warnings:

The use of the same nasal cannula on more than one patient poses a risk of cross-infection. As with all medical equipment, carefully route cables and connections to reduce the possibility of entanglement or strangulation.

Caution: U.S. federal law restricts this device to sale by or on the order of a licensed medical practitioner.

Note: The cannula is validated to give high quality flow signal for use in diagnosis of sleep disordered breathing. Using a cannula that has not been validated for this purpose might compromise the signal quality and the operational reliability of the system.

For distributor information go to: www.noxmedical.com
Assembled in Mexico